



EndoGuidance
SKIN STAPLER

CAK Disposables Skin Stapler



(888) 863 -1112 Ext. 422

6831 Edgewater Commerce Pkwy Ste 1101
Orlando, FL 32810

MedicalDisposables.us
SurgicalDivision@MedicalDisposables.us

HAVE QUESTIONS? PLEASE CALL (888) 863-1112

CAK Disposables Skin Stapler

Our disposable skin stapler is a lever action device with ergonomic design and strong construction that can be used to close superficial, straight surgical incisions and wounds.

It features a slim, lightweight body that fits most hand sizes. The unique angled head provides a clear view of the work site from your normal vantage point.

The unique angled head provides a clear view of the work site from your normal vantage point.



The clear side window lets you view the number of staples remaining to minimize wastage.

Each stapler is preloaded with 35 wide stainless-steel staplers.



CAK Disposables Skin Stapler Instructions

To use the skin stapler, first align it with the wound to be closed using the front arrow indicator as a guide.

Use your fingers to gently approximate the skin edges, then press the stapler down firmly on the skin while squeezing its handle to apply a staple.

Place a few evenly spaced staples (approximately 10mm apart) to ensure that full wound closure is supported.

A staple line is effective in preventing leakage of blood or air from the wound.

Each CAK Disposable Skin Stapler is individually sealed and sterilized. The device is intended for a single use only.